wiroBIGSWABBY

[de]	Produktdatenblatt WATTESTÄBCHEN	Seite 2
[en]	Product data sheet COTTON BUD	Page 6
[nl]	Productgegevensformulier WATTENSTAAFJES	Zijde 10
[fr]	Fiche produit COTON-TIGE	Page 14
[es]	Hoja de datos del producto BASTONCILLOS DE ALGODÓN	Página 18
[it]	Scheda prodotto TAMPONI	Pagina 22
[el]	Δελτίο δεδομένων προϊόντος ΒΑΜΒΑΚΟΦΟΡΟΙ ΣΤΥΛΕΟΙ	Σελίδα 26
[tr]	Ürün Bilgi Formu KULAK ÇUBUĞU	Sayfa 30
[ru]	Техническая спецификация БАТНАЯ ПАЛОЧКА	Сторона 34
[pl]	Karta produktu PATYCZKI HIGIENICZNE	Strona 38
[da]	Produktdatablad VATPIND	Side 42
[sv]	Produktdatablad BOMULLSPINNAR	Sida 46
[no]	Produktdatablad VATTPINNER	Side 50
[cs]	Informační list výrobku VATOVÉ TYČINKY	Strana 54
[hu]	Termékadatlap VATTAPÁLCIKA	Oldal: 58
[sk]	Informačný list výrobku VATOVÁ TYČINKA	Strana 62
[sl]	Podatkovni list izdelka VATIRANE PALČKE	Stran 66
[ro]	Fișa produsului BEȚIGAȘE PENTRU URECHI	Pagina 70
[pt]	Folha de dados do produto COTONETE	Página 74
[lt]	Gaminio duomenų lapas VATOS KRAPŠTUKAI	Puslapis 78
[lv]	Produkta datu lapa VATES KOCIŅI	Lappuse 82
[bg]	Информационен лист за продукта КЛЕЧКА С ПАМУК	Страница 86
[uk]	Технічний лист БАТНА ПАЛИЧКА	Сторінка 90
[et]	Toote teabeleht VATITIKUD	Lk 94
[fi]	Tuotetietolehti VANUPUIKOT	Sivu 98

wiroBIGSWABBY

WATTESTÄBCHEN

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Product data sheet

wiroBIGSWABBY

Manufacturer	
Company + Address	WIROS Wilfried Rosbach GmbH, Karl-Arnold-Str. 5, 47877 Willich, Germany
SRN	DE-MF-000004956
Trademark	care & serve®
Authorisation	
CH REP	Swiss AR Services AG, Industriestrasse 47, 6300 Zug, Switzerland
Product description	
Category	PREMIUM
Product description	care & serve® PREMIUM wiroBIGSWABBY Cotton bud Small cotton wool end Wooden stick
Product features	Small cotton wool end Wooden sticks
Material	
Material	Birchwood cotton
Application	
Intended use	Disposable product for general medical areas such as the targeted application of ointments or liquids, cleaning and much more.
Scope of application	Cleaning ears, nose and umbilical cord and treating wounds Extracting microbiological cultures (bacteria, viruses, fungi) Targeted application/removal of medications and substances
Application group	Use is restricted to appropriately trained professionals.
Patient group	Cotton bud suitable for all patient groups.
Indication	Cleaning: Cotton buds are regularly used for cleaning purposes in clinical practices. They are a useful tool for cleaning areas such as the ears, nose, umbilical cord and treating wounds. Taking swab samples: Cotton swabs are widely used for extracting microbiological cultures (bacteria, viruses, fungi). They are placed on or inside an infected area and then wiped across the breeding ground, such as an agar plate, where the bacteria grow from the swab. For certain microbiological tests, it is necessary to use sterilised cotton buds. Other purposes: Cotton buds are widely used for medical treatments and care. Among other things, they are used to apply medications to a specific area, remove certain substances or to apply cleaning substances such as Betadine. Examples include ointments for treating the nasal ostium, salve for the lips or ear canal, various types of liquid such as gentian violet on nail and skin lesions (from author's own experience).
Contraindication	Use in immunocompromised patients and premature infants. Non-sterile wood products should not be used for the collection, handling and processing of specimens for microbiological testing.
Notes on use	Disposable product Not sterile
Note	All serious incidents directly related to this product must be reported to the manufacturer and the competent authorities.
Application areas	Hospital Old people's home inc. specialist circles Medical practices Laboratory Cosmetics Wellness Beauty industry

Product data sheet

wiroBIGSWABBY

Supplementary material info Latex-free

Standards

CE marking Class I medical device in accordance with EU 2017/745

SRN DE-MF-000004956

Basis UDI DI 4051642-wiroBIGSWABBY-29

CE marking

The CE mark guarantees the free movement of products and goods within the European Economic Community. All products with the CE mark comply with the essential requirements of the European regulations specified.

VO (EU) 2023/988 Regulation (EU) No 2023/988 on general product safety

Use

Instructions for use Before use
Please ensure that the cotton bud has no chips or broken areas. Only use intact and unused cotton buds.

Use

Use the end of the bud to pick up the cleansing substance or substance to be applied and carefully spread or apply it on the area to be treated. Using a fresh cotton bud means the application is more accurate and hygienic than with your fingers.

Cotton buds must not be inserted into the ear canal. If cotton buds are used to clean the ears, only use them on the outer area of the ear with extreme care.

Safety notice Do not insert into the ear canal.

Disposal Unused and uncontaminated products can be incinerated in an environmentally friendly manner or disposed of at a waste disposal site. Contaminated products must be disposed of in accordance with the national laws and regulations valid for the respective type of contamination.

Shelf life 5 years from the production date, if unused and stored correctly.

Storage Keep out of direct sunlight.
Store in a dry place.
Incorrect storage can lead to a reduction in the shelf life.

Traceability Batch-related traceability from the determination of requirements to issuing.
Product data sheet Version 2 Valid from Batch 2128 +

Packaging

Recycling Licensed in accordance with VerpackG (German Packaging Act), Licence number DE1618662887322.
Please recycle the packaging.

www.wiros.de

WEB <https://www.wiros.de/shop/medizin-pflegebedarf/wattestaebchen/wirobigswabby/>

Variant 1

Sales unit 10000 Unit/Cardboard box

Packaging level 1 | 2 1000 Unit/Box

100 Unit/Polybag

Minimum order quantity 1 Cardboard box

Item numbers

Item number

Colour

End of the bud (Ø x L)

Stick (Ø x L)

85800302

natural

≈ 4 x 15 mm

≈ 2,2 x 230 mm

Product data sheet

wiroBIGSWABBY

GTINs	Item number	Unit of Use UDI	Packaging level 1 UDI	Packaging level 2 UDI	Cardboard box UDI
	85800302	4051642003863	4051642375069	4051642450070	4051642875064

The numerical data for the measurements, weights, thicknesses and tolerances are target values, and there may be some deviation.

The information represents the current state of our knowledge. It offers no guarantee of the product characteristics and does not form the basis of a contractual legal understanding. Please reconfirm any information that is older than 2 years.

Typographical and other errors excepted.

The product data sheets provided are the property of WIROS Wilfried Rosbach GmbH. WIROS Wilfried Rosbach GmbH is the owner of all copyrights and rights of use and exploitation for the product data sheets. It is not permitted to change the product data sheets or to use the pictograms. The product data sheets provided may only be used in conjunction with original WIROS products.

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