wiroCLEVER FLEX

[de]	Produktdatenblatt PP+PE KITTEL	Seite 2
[en]	Product data sheet PP+PE GOWN	Page 6
[nl]	Productgegevensformulier PP+PE JAS	Zijde 10
[fr]	Fiche produit BLOUSES PP+PE	Page 14
[es]	Hoja de datos del producto BATA DE PP+PE	Página 18
[it]	Scheda prodotto CAMICE IN PP+PE	Pagina 22
[el]	Δελτίο δεδομένων προϊόντος ΙΑΤΡΙΚΗ ΠΟΔΙΑ PP+PE	Σελίδα 26
[tr]	Ürün Bilgi Formu PP+PE ÖNLÜK	Sayfa 30
[ru]	Техническая спецификация ХАЛАТ ИЗ ПП+ПЭ	Сторона 34
[pl]	Karta produktu KITEL PP+PE	Strona 38
[da]	Produktdatablad PP + PE-KITTEL	Side 42
[sv]	Produktdatablad PP+PE-SKYDDSRÖCK	Sida 46
[no]	Produktdatablad PP+PE-KITTEL	Side 50
[cs]	Informační list výrobku PP+PE PLÁŠŤ	Strana 54
[hu]	Termékadatlap PP+PE LÁTOGATÓI KÖPENY	Oldal: 58
[sk]	Informačný list výrobku PP + PE PLÁŠŤ	Strana 62
[sl]	Podatkovni list izdelka PP+PE HALJA	Stran 66
[ro]	Fișa produsului HALAT PP+PE	Pagina 70
[pt]	Folha de dados do produto BATA PP+PE	Página 74
[lt]	Gaminio duomenų lapas PP+PE DARBINIAI CHALATAI	Puslapis 78
[lv]	Produkta datu lapa PP+PE UZSVĀRČĪ	Lappuse 82
[bg]	Информационен лист за продукта PP+PE ПРЕСТИЛКА	Страница 86
[uk]	Технічний лист ХАЛАТ З ПП+ПЕ	Сторінка 90
[et]	Toote teabeleht PP+PE KITTEL	Lk 94
[fi]	Tuotetietolehti PP + PE -TAKIT	Sivu 98

wiroCLEVER FLEX

PP+PE KITTEL

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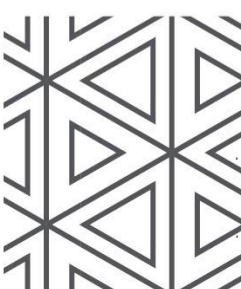
wiroCLEVER FLEX

PP+PE GOWN

PE coating on arms & front | Elasticated cuffs | Neck & waist ties | 2 cut-off perforations (for shortening the length)

PREMIUM+

Product data sheet

wiroCLEVER FLEX

Manufacturer

Company + Address	WIROS Wilfried Rosbach GmbH, Karl-Arnold-Str. 5, 47877 Willich, Germany
SRN	DE-MF-000004956
Trademark	care & serve®

Authorisation

CH REP	Swiss AR Services AG, Industriestrasse 47, 6300 Zug, Switzerland
--------	--

Product description

Category	PREMIUM+
Product description	care & serve® PREMIUM+ wiroCLEVER FLEX PP+PE gown PE coating on arms & front Elasticated cuffs Neck & waist ties 2 cut-off perforations (for shortening the length)
Product features	PE coating on arms & front Elasticated cuffs Neck & waist ties 2 cut-off perforations (for shortening the length)

Material

Material	Polypropylene spun-bonded fabric with polyethylene coating
Material weight	PP: $\approx 20 \text{ g/m}^2 (\pm 2 \text{ g/m}^2)$

Application

Intended use	Medical protective clothing as defined in the Technical Rules for Biological Agents (TRBA 250) to prevent contamination of work or normal clothes during interventions or care measures resulting from direct contact with blood, secretions, excretions or other contaminated material.
Contact with food produce	Short-term indirect contact with foodstuffs is permitted.
Scope of application	There is no direct medical application on the patient. Medical protective clothing is intended to cover clothing and/or hair.
Application group	Employees and visitors from laboratories, the pharmaceutical industry, hospitals or similar institutions.
Patient group	Medical protective clothing is suitable for all patient groups.
Indication	Medical clothing as a barrier to cover clothing and/or hair.
Contraindication	No known contraindications.
Notes on use	Disposable product Not sterile
Limitation	No PPE (personal protective equipment) Not suitable for packing or storing food.
Note	All serious incidents directly related to this product must be reported to the manufacturer and the competent authorities.
Application areas	Hospital Old people's home inc. specialist circles Industry Pharmaceutical industry Laboratory Food processing HACCP Cleaning Hygiene

Product data sheet

wiroCLEVER FLEX

Supplementary material info	Latex-free	
Health advice	If skin irritation or allergic reactions occur, discontinue use immediately and seek medical advice. As far as is known according to the current state of the art, the products do not contain any toxic, carcinogenic, mutagenic or reproductively toxic substances when used in accordance with their intended use. These products do not contain latex components and are considered hypoallergenic. However, they can cause irritation in sensitive skin. Long-term wear may restrict skin ventilation, which can cause skin irritation.	

Standards

CE marking	Class I medical device in accordance with EU 2017/745	 
SRN	DE-MF-000004956	
Basis UDI DI	4051642-wiroCLEVERFLEX-X5	
	CE marking The CE mark guarantees the free movement of products and goods within the European Economic Community. All products with the CE mark comply with the essential requirements of the European regulations specified.	
VO (EU) 2023/988	Regulation (EU) No 2023/988 on general product safety	
VO (EG) 1935/2004	Regulation regarding plastic materials and articles intended to come into contact with food.	
VO (EU) 10/2011	Regulation on plastic materials and articles intended to come into contact with food	

Use

Instructions for use	Before use Please make sure that you have selected the right product for your needs and areas of use. Please do not wear medical protective clothing if it is faulty or has already been used. Medical protective clothing is intended for short-term use and use is limited to one patient in the medical field. Use Gowns are worn with the closed side at the front and are fastened at the back or side. To do this, slide your arms into the sleeves and fasten the ties on your neck and waist. In order to prevent contamination with clothing when taking the gown off, it is recommended for a second person to untie the gown from behind and slide it forwards over your shoulders. The person wearing the gown grasps the upper ends of the gown and turns it inside out when removing it.	
Safety notice	According to our risk analysis under EN ISO 14971:2019 + A11:2021, no additional safety instructions are required for this product.	
Disposal	Unused and uncontaminated products can be incinerated in an environmentally friendly manner or disposed of at a waste disposal site. Contaminated products must be disposed of in accordance with the national laws and regulations valid for the respective type of contamination.	
Shelf life	5 years from the production date, if unused and stored correctly.	
Storage	Keep out of direct sunlight. Store in a dry place. Incorrect storage can lead to a reduction in the shelf life.	 
Traceability	Batch-related traceability from the determination of requirements to issuing. Product data sheet Version 2 Valid from Batch 2617 +	

Packaging

Recycling	Licensed in accordance with VerpackG (German Packaging Act), Licence number DE1618662887322. Please recycle the packaging.	
-----------	---	---

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The numerical data for the measurements, weights, thicknesses and tolerances are target values, and there may be some deviation.

The information represents the current state of our knowledge. It offers no guarantee of the product characteristics and does not form the basis of a contractual legal understanding. Please reconfirm any information that is older than 2 years. Typographical and other errors excepted.

The product data sheets provided are the property of WIROS Wilfried Rosbach GmbH. WIROS Wilfried Rosbach GmbH is the owner of all copyrights and rights of use and exploitation for the product data sheets. It is not permitted to change the product data sheets or to use the pictograms. The product data sheets provided may only be used in conjunction with original WIROS products.

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